

**3M** Science.  
Applied to Life.™

# The ultimate varnish for every generation.

**Vanish™**  
5% Sodium Fluoride  
White Varnish with  
Tri-Calcium Phosphate



## Summary of advantages

- The #1-selling fluoride varnish in the U.S.
- Contains exclusive 3M calcium and phosphate ingredient
- Contains 22,600 ppm fluoride
- Adheres to teeth for continuous fluoride contact
- Provides a steady, sustained release of fluoride
- Covers more tooth surfaces to deliver fluoride
- Easy to apply to moist teeth
- Relieves hypersensitivity
- Contains xylitol
- Over 8 years of clinical success

## 3M's exclusive TCP offers more protection.

Vanish™ 5% Sodium Fluoride White Varnish with Tri-Calcium Phosphate (TCP) is a proprietary formula that contains fluoride, calcium and phosphate—components naturally found in saliva. It delivers targeted and sustained fluoride and calcium release.


Vanish white varnish flows across tooth surfaces and comes in contact with saliva, releasing fluoride AND calcium over the life of the varnish—up to 24 hours!

### Flows better.

Clinical studies show that Vanish white varnish flows to more than twice as many tooth surfaces after application.

### Migration of Varnish

(Mean +/- SEM; n=19)

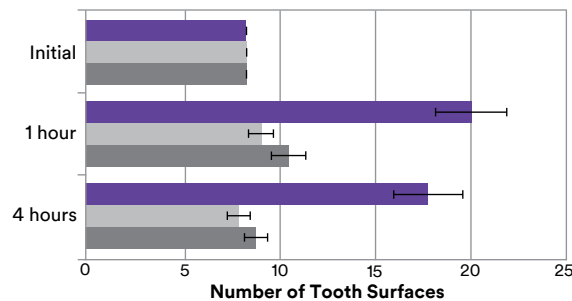

■ Vanish™ White Varnish ■ Competitive Varnish ■ Competitive Varnish

Vanish™ 5% Sodium Fluoride White Varnish or the competitive varnishes were applied to 8 anterior maxillary surfaces in vivo. Migration of the varnishes was measured at 1 hour and 4 hours. Source: 3M internal data

## Steady fluoride release.

Unlike varnishes that release most of their fluoride in the first hour, Vanish white varnish provides a prolonged, steady delivery of fluoride, allowing absorption into the tooth.


Independent studies have shown the value of sustained release of fluoride from varnishes:

**“Most studies showing a higher efficacy of fluoride varnish attribute it to its increased contact time.”\***

\*Source: Khattak MF, et al. Comparison of three topical fluorides using computer imaging. J Clin Pediatr Dent 2005; 30(2): 139-144.

### Cumulative Fluoride Release

(± 95% confidence interval; 3 lots, n=5 each)

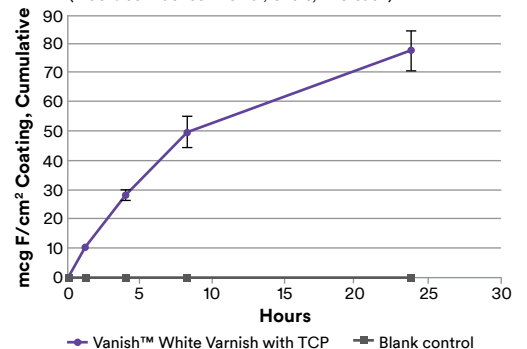

Vanish™ 5% Sodium Fluoride White Varnish delivers a steady release of fluoride throughout the wear time. Source: 3M internal data

## Easy to apply.

Simply apply one thin coat, sweeping the brush across tooth surfaces—and the Vanish™ 5% Sodium Fluoride White Varnish formula does the rest.

- Sets in the presence of saliva—no drying needed.
- Enhanced flow characteristics allow Vanish white varnish to reach interproximal areas other varnishes may miss.
- Covers and adheres to the teeth until brushed off.



Apply Vanish™ 5% Sodium Fluoride White Varnish evenly in a thin layer over treatment area(s) with sweeping, horizontal brush strokes. No suction required.

## Protection you can't see.

Vanish white varnish is virtually invisible on the teeth—but leaves a temporary thin coating so patients know it's working.



Immediately after Vanish™ 5% Sodium Fluoride White Varnish was applied.

## Ordering Information

Item #	Product Information
12149C	<b>Vanish™ 5% Sodium Fluoride White Varnish 50 Pack—Cherry Flavor</b> Includes: 50 unit-dose packets (0.5ml each) and applicator brushes in individual foil pouches; 60 dosage stickers; Technique Card
12149L	<b>Vanish™ 5% Sodium Fluoride White Varnish 50 Pack—Melon Flavor</b> Includes: 50 unit-dose packets (0.5ml each) and applicator brushes in individual foil pouches; 60 dosage stickers; Technique Card
12149M	<b>Vanish™ 5% Sodium Fluoride White Varnish 50 Pack—Mint Flavor</b> Includes: 50 unit-dose packets (0.5ml each) and applicator brushes in individual foil pouches; 60 dosage stickers; Technique Card
12150C	<b>Vanish™ 5% Sodium Fluoride White Varnish 100 Pack—Cherry Flavor</b> Includes: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card
12150L	<b>Vanish™ 5% Sodium Fluoride White Varnish 100 Pack—Melon Flavor</b> Includes: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card
12150M	<b>Vanish™ 5% Sodium Fluoride White Varnish 100 Pack—Mint Flavor</b> Includes: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card
12151C	<b>Vanish™ 5% Sodium Fluoride White Varnish Bulk Pack—Cherry Flavor</b> Includes: 10 boxes. Each box contains: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card
12151L	<b>Vanish™ 5% Sodium Fluoride White Varnish Bulk Pack—Melon Flavor</b> Includes: 10 boxes. Each box contains: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card
12151M	<b>Vanish™ 5% Sodium Fluoride White Varnish Bulk Pack—Mint Flavor</b> Includes: 10 boxes. Each box contains: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card
12151CL	<b>Vanish™ 5% Sodium Fluoride White Varnish Bulk Pack—Cherry &amp; Melon Flavors</b> Includes: 10 boxes (5 Cherry, 5 Melon). Each box contains: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card
12151X	<b>Vanish™ 5% Sodium Fluoride White Varnish Bulk Pack—Assorted Flavors</b> Includes: 10 boxes (4 Cherry, 4 Melon, 2 Mint). Each box contains: 100 unit-dose packets (0.5ml each); 100 applicator brushes; 2 patient pads; 108 dosage stickers; Technique Card



3M Oral Care  
2510 Conway Avenue  
St. Paul, MN 55144-1000 USA  
Phone 1-800-634-2249  
Web 3M.com/dental

[www.3M.com/PreventiveCare](http://www.3M.com/PreventiveCare)

3M Canada  
Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
Phone 1-888-363-3685

3M, ESPE and Vanish are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Printed in USA. © 3M 2016. All rights reserved. 70-2013-0391-7